

# NO MORE ROCKING OR SEESAW-EFFECTS

## quinto

the third hand

the quinto is a very precise yet simple  
solution for fixing even the most complex  
dental restoration on the model


# NO MORE ROCKING OR SEESAW- EFFECTS


## uses

- secures even the most complex restorations simply and precisely on the model
- holds multiple pieces of a sectioned bridge in place for welding
- helps adhering an electroformed gold structure produced over an implant bar to the tertiary superstructure of the denture base
- makes it easy to position a cobalt chrome denture base extension
- ensures centric fixing of a pair of models before mounting on an articulator

## advantages

- eliminates the chance of rocking or seesawing on all different cases
- keeps both hands free for holding or adjustments
- distortion-free uniform hardening of sticky wax or hot melt adhesive
- no problems with mounting on an articulator
- no waiting while materials set, the technician is free for other tasks.
- saves time and increases productivity
- individually adjustable to fit all types of restorations
- can be directly grounded with the phaser welding unit

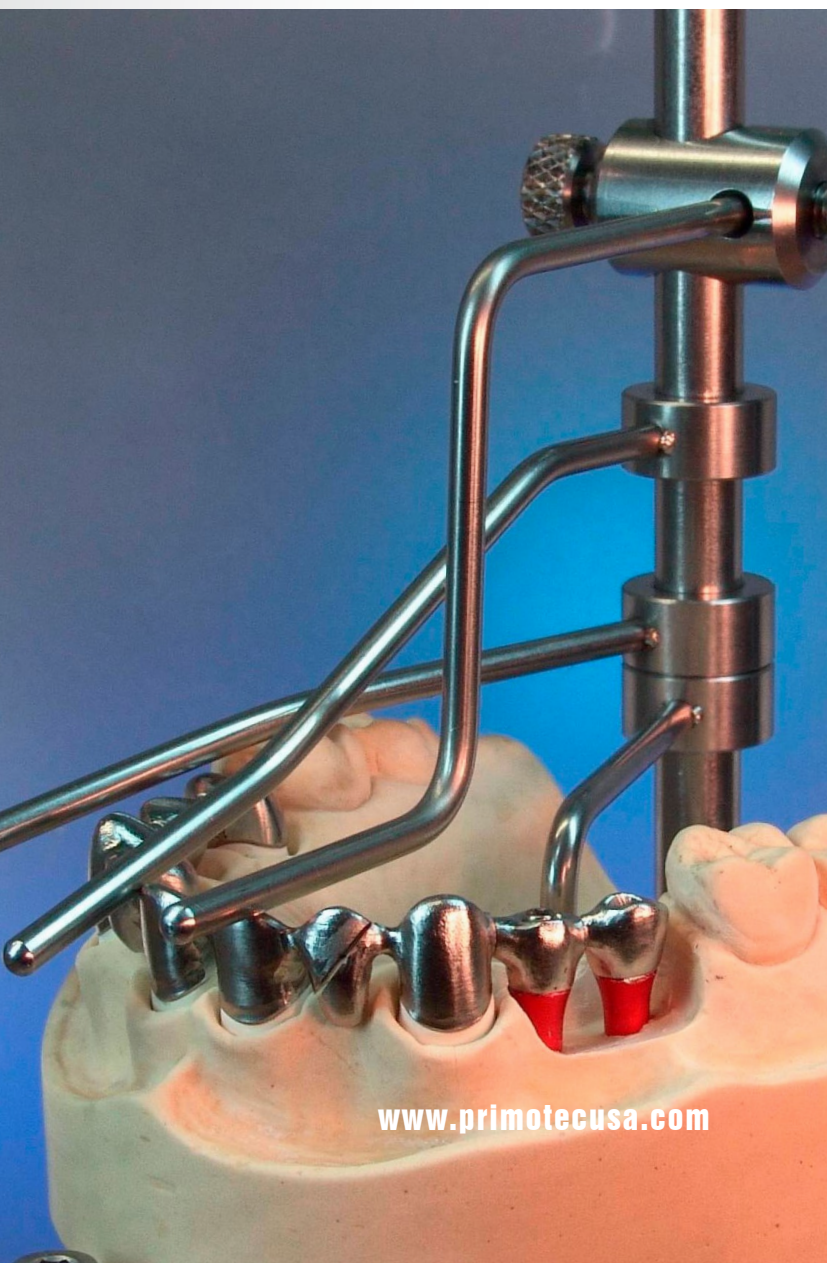


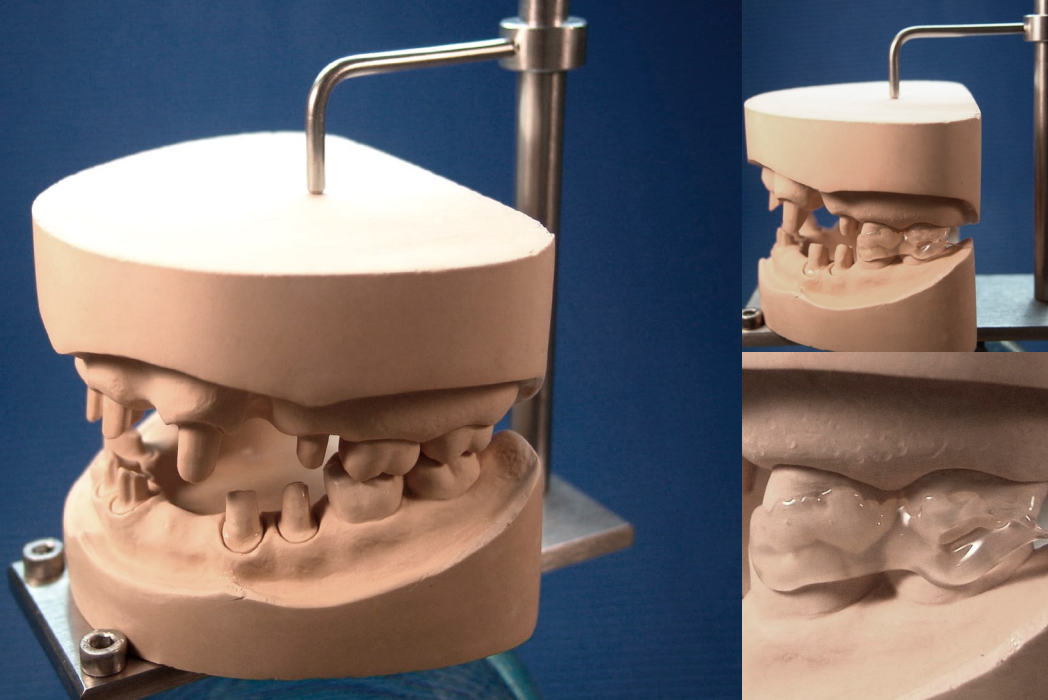

## working procedure



**1**  
place the model on the Quinto base and press the small 90° angle arm against the model securing the model to the base

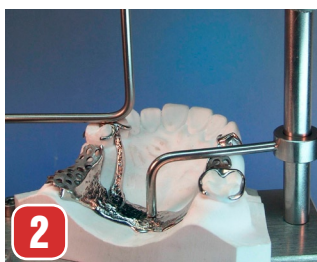


**2**  
depending on the type of restoration, use any of the other arms to secure the sections on the model



**3**  
now you can weld, glue or do anything with the case and have both hands free.

## ORDERING INFO

Item #		Item	Price
QU100		<b>Quinto</b> - model base with vertical holder bar - clip-arm 1, fixing the model - clip-arm 2, one flat angle - clip-arm 3, two flat angles - screw-joint-arm 1, one 90 angle - screw-joint-arm 2, two 90 angles	\$349.00



# no more rocking or seesaw-effects

5 differently angled arms allow for holding any type of case perfectly in place



Primotec USA  
25 Sylvan Road South, Suite N  
Westport, CT 06880  
866-643-3129

[info@primotecusa.com](mailto:info@primotecusa.com)  
[www.primotecusa.com](http://www.primotecusa.com)