

# THE BITE SPLINT WITH THERMO-EFFECT



**excited  
patients**

**pleased  
dentists**

**relaxed  
technicians**

## PREMIOtemp CLEAR FLEX

**Nothing presses, nothing straints**

**the milled PREMIOtemp CLEAR FLEX bite  
splint is just super comfortable to wear.**

## PATIENT INFORMATION



# CLEAR FLEX

# MAXIMUM BREAKAGE RESISTANCE WITH THERMO-EFFECT

PREMIO



**Maximum breakage resistance** at body temperature through thermoplastic flexibility

## TOP FEATURES

## YOUR BENEFITS



**Memory-Effect:** 100% reset at body temperature without loss of surface hardness

- Tension-free wearing comfort and precise fitting due to the thermo-effect
- Very durable – maximum breakage resistance through memory effect – more resistant to breakage than most conventional PMMA plastics
- High transparency for almost invisible wearing
- Very low plaque affinity

# CLEAR FLEX

# CLEAR FLEX

The milled **PREMIOtemp**  
**CLEAR FLEX** bite splint from **primotec**

## Care instructions:

- The PREMIOtemp CLEAR FLEX splint should be cleaned under running water with toothpaste after each use
- Store your bite splint in a dry and dark place if not worn
- Do not use denture cleaners /alcohol-based cleaners
- Place the bite splint in warm water at 40-50 °C for 1-2 minutes before insertion
- Protect the splint from extreme heat and coldness



Primotec USA  
360 Westport Ave.  
Norwalk, CT 06851  
203-682-6429  
[www.primotecusa.com](http://www.primotecusa.com)  
[info@primotecusa.com](mailto:info@primotecusa.com)