

THE BITE SPLINT WITH THERMO-EFFECT

 **primotec**[®]
DIGITAL

**excited
patients**

**pleased
dentists**

**relaxed
technicians**

PREMIOtemp CLEAR FLEX

Nothing presses, nothing straints

**the milled PREMIOtemp CLEAR FLEX bite
splint is just super comfortable to wear.**

PATIENT INFORMATION



CLEAR FLEX

MAXIMUM BREAKAGE RESISTANCE WITH THERMO-EFFECT

PREMIO



Maximum breakage resistance at body temperature through thermoplastic flexibility

TOP FEATURES

YOUR BENEFITS



Memory-Effect: 100% reset at body temperature without loss of surface hardness

- Tension-free wearing comfort and precise fitting due to the thermo-effect
- Very durable – maximum breakage resistance through memory effect – more resistant to breakage than most conventional PMMA plastics
- High transparency for almost invisible wearing
- Very low plaque affinity

CLEAR FLEX

CLEAR FLEX

The milled **PREMIOtemp**
CLEAR FLEX bite splint from **primotec**

Care instructions:

- The PREMIOtemp CLEAR FLEX splint should be cleaned under running water with toothpaste after each use
- Store your bite splint in a glass of water if not worn
- Do not use denture cleaners /alcohol based cleaners
- Place the bite splint in warm water at 40-50 °C for 1-2 minutes before insertion
- Protect the splint from extreme heat and coldness



Primotec USA
360 Westport Ave.
Norwalk, CT 06851
203-682-6429
www.primotecusa.com
info@primotecusa.com